



All-in-One product with a Cotton Feel backsheet

- An **All-in-One system**, with a Cotton Feel backsheet
- **Anti-leak cuffs** to protect against leakage
- The **Wetness indicators** act as a guide to easily show when a pad needs changing
- Contains an **odour control system**
- Resealable tapes for a **perfect fit**



iD Expert Slip is also available with a Plastic backsheet (selected products only)

iD Expert Slip Cotton Feel

Product Code	Product Name	Size	Droplet Scale	ISO / Total Absorbency	Colour	Waist Size	Measurements	Pieces per pack	Pieces per pack	Pieces per pack
510255280	Slip Normal	M	▲▲▲	1850ml	■	80-125cm	800X680mm	28	4	112
5610355280	Slip Normal	L	▲▲▲	1900ml	■	115-55cm	980X800mm	28	3	84
5630160140	Slip Plus	S	▲▲▲	100ml	■	50-90cm	700X530mm	14	4	56
5610260280	Slip Plus	M	▲▲▲	2175ml	■	80-125cm	800X680mm	28	4	112
5610360280	Slip Plus	L	▲▲▲	2400ml	■	115-55cm	980X800mm	28	3	84
5630460140	Slip Plus	XL	▲▲▲	3200ml	■	120-170cm	980X850mm	14	4	56
5630165140	Slip Extra	S	▲▲▲	1950ml	■	50-90cm	700X530mm	14	4	56
5610265280	Slip Extra	M	▲▲▲	2400ml	■	80-125cm	800X680mm	28	4	112
5610365280	Slip Extra	L	▲▲▲	2850ml	■	115cm-155cm	980X800mm	28	2	56
5630465140	Slip Extra	XL	▲▲▲	3400ml	■	120-170cm	980X850mm	14	4	56
5610270280	Slip Extra Plus	M	▲▲▲	2750ml	■	80-125cm	800X680mm	28	4	112
5610370280	Slip Extra Plus	L	▲▲▲	2950ml	■	115cm-155cm	980X800mm	28	2	56
5630075140	Slip Super	XS	▲▲▲	1550ml	■	40-70cm	620X410mm	14	12	168
5630175140	Slip Super	S	▲▲▲	2100ml	■	50-90cm	700X530mm	14	4	56
5630275280	Slip Super	M	▲▲▲	3600ml	■	80-125cm	800X680mm	28	3	56
5630375280	Slip Super	L	▲▲▲	4100ml	■	115-155cm	980X800mm	28	3	56
5630475140	Slip Super	XL	▲▲▲	3800ml	■	120-170cm	980X850mm	14	4	56
5630180200	Slip Maxi	S	▲▲▲	2300ml	■	50-90cm	700X530mm	20	4	80
5630280150	Slip Maxi	M	▲▲▲	3700ml	■	80-125cm	800X680mm	15	3	45
5630380150	Slip Maxi	L	▲▲▲	4500ml	■	115-155cm	980X800mm	15	3	45

Fitting Guide

An All-In-One offers an effective solution for managing moderate to heavy incontinence, with a high level of absorption and range of sizes.

Ensure the correct size by measuring the waist. Measure the user, never assume the measurement. Remember, bigger is not better! Then choose the correct absorbency.

Gently open out and then fold in half lengthways with the absorbent area of the pad facing inwards.

Do not shake or twist the pad as this can damage it and reduce the efficacy of the product.

Standing

Keeping the pad folded, pass the pad between the legs in the most appropriate way for the user. Open the pad out over the buttocks and lower abdomen, ensuring the side with the fixation tapes is placed at the back. Align with the waist and centre correctly using the wetness indicators as a guide.

Make sure the soft elastics are fitted closely at the groin. Secure the re-sealable tapes; fit the top tapes first, or dependant on the user, you may need to fit the bottom tapes first. The position of the tapes is wholly dependent on the user needs.

It is important to create a seal around the legs and groin to reduce risk of leakage. To do this the bottom tapes will probably need to be secured in an angled position. The choice of upward or downward positioning of tapes is determined by the shape of the client, mainly around the buttocks.



Lying Down

With the user lying on their side, keep the pad folded and pass between the legs in the most appropriate way for the user. Open the pad out over the buttocks and lower abdomen, ensuring the side with the fixation tapes is placed at the back. Align with the waist and centre correctly using the wetness indicators as a guide.

Secure the re-sealable tapes; fit the top tapes first, or dependant on the user, you may need to fit the bottom tapes first. The position of the tapes is entirely dependent on the patient needs. Fasten the re-sealable tapes on the upper-most side, and then gently roll them and fix the tapes on the opposite side. Now position the patient on to their back and ensure the anti-leak cuffs are fitted close at the groin.

The tapes can then be readjusted as necessary. Remember it is important to create a seal around the legs and groin to reduce risk of leakage. To do this the bottom tapes will probably need to be secured in an angled position. The choice of upward or downward positioning of tapes is determined by the shape of the user, mainly around the buttocks.

Always ensure the tapes are secured to the pad and are not lying on, or close to the skin, as this may cause irritation and discomfort. When carrying out routine position changes remember to adjust the tapes accordingly to ensure leg seals are maintained.



People who bought iD Expert Belt also bought...



iD Care No Rinse Cleansing Milk



iD Care No Rinse Body Cleansing Foam



iD Care Wet Wipes



All-in-One product with a plastic backsheet

- An **All-in-One system**, with a plastic backsheet
- **Anti-leak cuffs** to protect against leakage
- The **Wetness indicators** act as a guide to easily show when a pad needs changing
- Contains an **odour control system**
- Resealable tapes for a **perfect fit**

iD Expert Slip is also available with a **Cotton Feel backsheet** (selected products only)



Fitting Guide

An All-In-One offers an effective solution for managing moderate to heavy incontinence, with a high level of absorption and range of sizes.

Ensure the correct size by measuring the waist. Measure the user, never assume the measurement. Remember, bigger is not better! Then choose the correct absorbency.

Gently open out and then fold in half lengthways with the absorbent area of the pad facing inwards.

Do not shake or twist the pad as this can damage it and reduce the efficacy of the product.

Standing

Keeping the pad folded, pass the pad between the legs in the most appropriate way for the user. Open the pad out over the buttocks and lower abdomen, ensuring the side with the fixation tapes is placed at the back. Align with the waist and centre correctly using the wetness indicators as a guide.

Make sure the soft elastics are fitted closely at the groin. Secure the re-sealable tapes; fit the top tapes first, or dependant on the user, you may need to fit the bottom tapes first. The position of the tapes is wholly dependent on the user needs.

It is important to create a seal around the legs and groin to reduce risk of leakage. To do this the bottom tapes will probably need to be secured in an angled position. The choice of upward or downward positioning of tapes is determined by the shape of the client, mainly around the buttocks.



Lying Down

With the user lying on their side, keep the pad folded and pass between the legs in the most appropriate way for the user. Open the pad out over the buttocks and lower abdomen, ensuring the side with the fixation tapes is placed at the back. Align with the waist and centre correctly using the wetness indicators as a guide.

Secure the re-sealable tapes; fit the top tapes first, or dependant on the user, you may need to fit the bottom tapes first. The position of the tapes is entirely dependent on the patient needs. Fasten the re-sealable tapes on the upper-most side, and then gently roll them and fix the tapes on the opposite side. Now position the patient on to their back and ensure the anti-leak cuffs are fitted close at the groin.

The tapes can then be readjusted as necessary. Remember it is important to create a seal around the legs and groin to reduce risk of leakage. To do this the bottom tapes will probably need to be secured in an angled position. The choice of upward or downward positioning of tapes is determined by the shape of the user, mainly around the buttocks.

Always ensure the tapes are secured to the pad and are not lying on, or close to the skin, as this may cause irritation and discomfort. When carrying out routine position changes remember to adjust the tapes accordingly to ensure leg seals are maintained.



iD Expert Slip plastic backsheet

Product Code	Product Name	Size	Droplet Scale	ISO / Total Absorbency	Colour	Waist Size	Measurements	Pieces per pack	Pieces per pack	Pieces per pack
5600255280	Slip Normal	M	3 dark, 3 light	1850ml	Black	80-125cm	800x680mm	28	4	112
5600355280	Slip Normal	L	3 dark, 3 light	1900ml	Black	115-155cm	980x800mm	28	3	84
5600260280	Slip Plus	M	4 blue, 2 light	2175ml	Blue	80-125cm	800x680mm	28	4	112
5600360280	Slip Plus	L	4 blue, 2 light	2400ml	Blue	115-155cm	980x800mm	28	3	84
5600265280	Slip Extra	M	5 purple, 1 light	2400ml	Purple	80-125cm	800x680mm	28	4	112
5600365280	Slip Extra	L	5 purple, 1 light	2850ml	Purple	115-155cm	980x800mm	28	2	56
5600270280	Slip Extra Plus	M	6 yellow, 1 light	2750ml	Yellow	80-125cm	800x680mm	28	4	112
5600370280	Slip Extra Plus	L	6 yellow, 1 light	2950ml	Yellow	115-155cm	980x800mm	28	2	56
5620275280	Slip Super	M	7 green, 1 light	3600ml	Green	80-125cm	800x680mm	28	2	56
5620375280	Slip Super	L	7 green, 1 light	4100ml	Green	115-155cm	980x800mm	28	2	56

People who bought iD Expert Belt also bought...



iD Care No Rinse Cleansing Milk



iD Care No Rinse Body Cleansing Foam



iD Care Wet Wipes