


## How to care for Bed pads



**Before** using this product please read and follow our 'How to Care For Your Product Guide' featured below.

Bed pads are manufactured using the latest textile technology and are constructed from three different fabrics. We have allowed for shrinkage of the fabrics in the manufacturing process and this affects the products initial appearance. You may notice that the top layer of the absorbent products is slightly bigger than the layers beneath it. The product will become **smoother** and be more **comfortable** if you follow these instructions **BEFORE USE**.

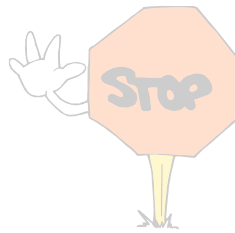

Wash the product **BEFORE USE** to make it smooth and comfortable.



Choose the boil wash cycle that has the **HIGHEST TEMPERATURE**.



Use a washing powder that **DOES NOT** contain **Fabric Softeners** or **Cationic Surfactants**, as these will reduce the absorbency of the product.



**DO NOT** add **Fabric Conditioners, Softeners** or products that contain **Cationic Surfactants**, as these will reduce the absorbency of the product.



You may dry the product in a Tumble Dryer provided the temperature **DOES NOT** exceed 77°C (medium setting) **and the product is allowed to cool prior to folding.**



You may dry the product on a heated radiator provided that the waterproof layer **DOES NOT** come into contact with the direct heat of the radiator.



If drying outside peg the product on the line so that the fabric layer is outer most.

### Gompels HealthCare Ltd

No. 1 Swift Way, Bowerhill Ind. Est. Melksham, Wilts. SN12 6GX

Registered Co. 4416138 in England at above address

Tel: 0845 450 2420 Fax: 0845 450 2422 Email: sales@gompels.co.uk

